


# Urine Collection Kit

## This Kit Includes:




1 collection tube

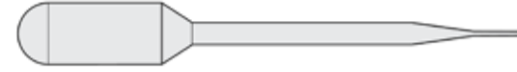

1 pipette



1 collection cup

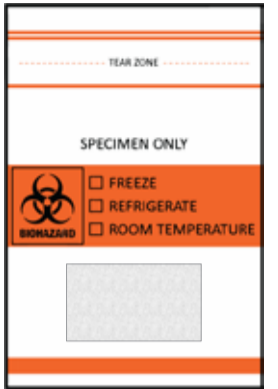


1 specimen bag  
(with absorbent pad)

## READ FIRST

1. Read all instructions thoroughly before collecting your specimens.
2. Be sure to register your kit
3. Do not drink alcohol on the day of collection.
4. Do not collect specimens during menstruation.
5. Collection tubes may contain a liquid preservative. **DO NOT EMPTY LIQUID PRESERVATIVE FROM THE COLLECTION TUBES IF PRESENT.**

## TO PREVENT REJECTED SPECIMENS, PLEASE READ ALL INSTRUCTIONS BEFORE BEGINNING COLLECTION

1.

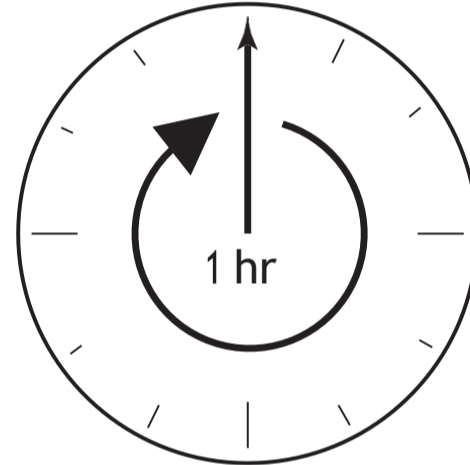

Do not urinate for at least 1 hour prior to collection. Using first morning urine is best.

2.




Urinate directly into the collection cup, filling between 1/3 and 1/2.


3.



Pour from the collection cup or use the pipette to transfer urine from the collection cup into the collection tube with a **YELLOW CAP**.

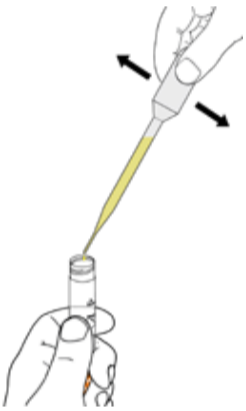


4.

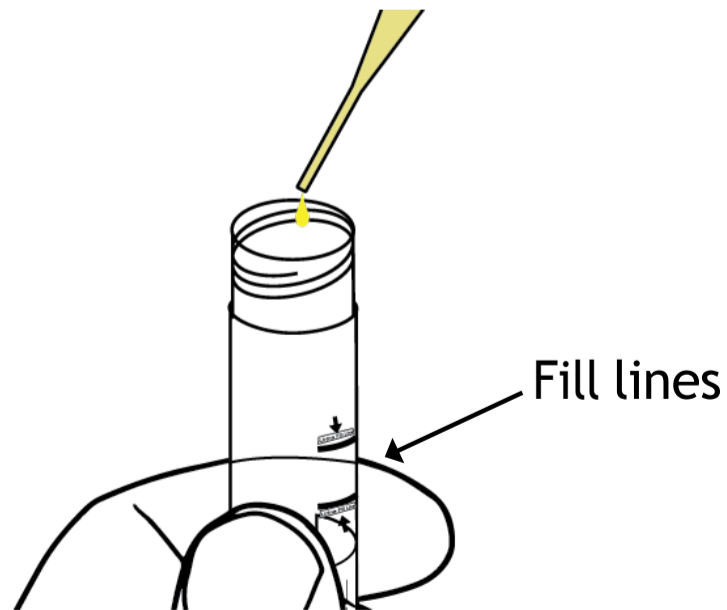

Fill lines


Fill the collection tube(s) until the combined liquid is between the designated fill lines. **Do not fill tube past the maximum fill line or specimen will be rejected.**

5.

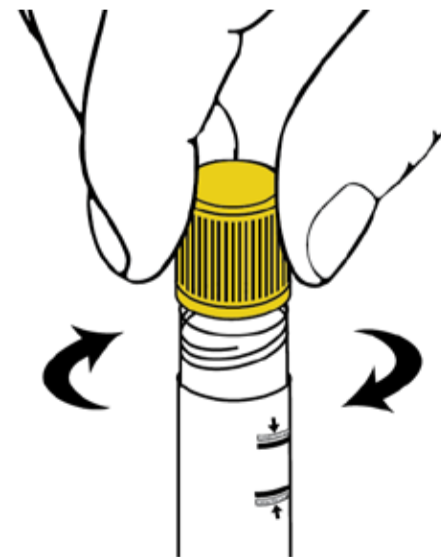


Place lid onto the collection tube and ensure it is closed evenly and tightly.

6.

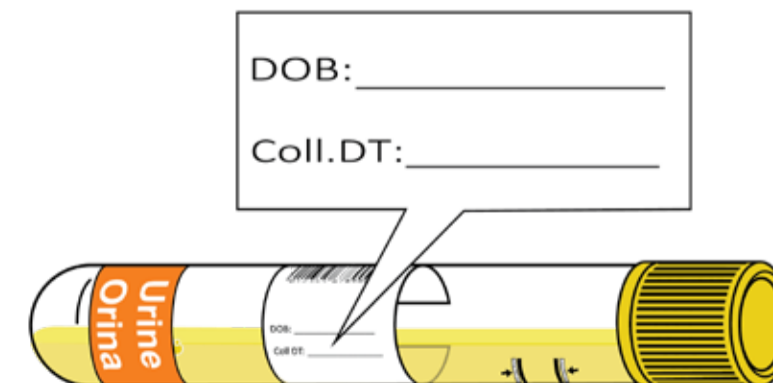

Write your date of birth and the specimen collection date (today's date) in MM/DD/YYYY format onto the tube in the designated areas.

7.

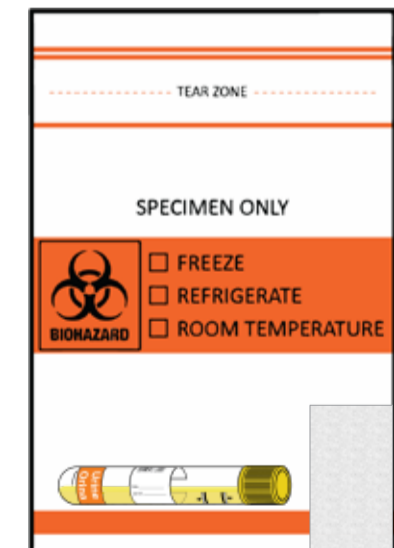

Place specimen into the biohazard bag with the absorbent pad and ensure the seal is closed completely. Now place the sample back into the kit box. Place the box in the FedEx envelope. Affix the return label and drop off at an authorized FedEx shipping location.